**Carraca Dinamométrica 10-70Ncm**  
**Clé a Cliquet Dynamométrique 10-70Ncm**  
**Torque Wrench 10-70Ncm**  
**Chiave dinamometrica 10-70 Ncm**  
**Chave Dinamométrica 10-70 Ncm**  
**Drehmomentschlüssel 10-70 Ncm**

PROSPDIN\_rev001  
2018.04.05

Phibo Headquarters  
Polígono Industrial Mas d'en Cisa, Gato Pérez, 3  
08181 | Sentmenat | Barcelona | Spanien | [www.phibo.com](http://www.phibo.com)

# Carraca Dinamométrica 10-70 Ncm

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Lubricant

## Description of the torque wrench

The torque wrench with adjustable force is a dental device used to tighten or loosen screws, prosthesis components and implants. It is a precision instrument that can be disassembled and that is supplied unsterile. To ensure that it functions perfectly every time, the torque wrench must be disassembled, disinfected, cleaned, greased and sterilised after each use, according to the instructions for use. Persons who use this device are obliged to read the instructions for use prior to handling.

The handling and the use of the product are carried out without direct control from our side and remain under responsibility of the user. The user is liable for any possible damage that could occur.

Before each use, in order to guarantee high torque precision, the device must be checked upon it's functioning.

This instrument is not a measuring device.

## Use

By turning the torque adjustment screw, the torque wrench can be set to the desired torque value. To set the torque value correctly, the torque adjustment screw must be turned clockwise to reach the required torque values and set to the exact line marking. Ensure that the line on the handle is in straight alignment with the line on the torque adjustment screw. In order to change from a higher to a lower torque value, one must screw two turns under desired torque value, then screw clockwise again to the exact line marking. Ratchet mode could be set by turning the torque adjustment screw until the R marking.

The word "IN" on the cover (3), shows the position if the wrench that is used for tightening; by turning the device over, the word "OUT" is used for loosening screws.

## Grease

"Instrument lubricant" approved USDA H1.

## Precision over lifetime

±10% of maximum torque

## Lifetime

3 years after supply date of manufacture

## Recommendations

This instrument must not be used for any applications other than those listed in the section "Description of the torque wrench" or with equipment that could damage the intended use of the device.

The persons in charge for the use and maintenance of this material instrument should monitor any deterioration of the tightening, ratchet and torque mechanism of the device and, in the event of a defect, return wrench to the supplier.

During assembly, it is essential not to mix the various components belonging to different torque wrenches because the components are not interchangeable.

If a component is lost, please return the whole instrument immediately to your retailer for repair. Components cannot be sold separately

Do not store the wrench with the spring compressed but with the torque set to its minimum.

This device must not be sterilised in the packaging provided by the manufacturer.

## Validity

This document cancels and replaces all previous versions.


The specifications contained herein can change without warning.

The official version of this documents is the version in Spanish.

## Individual parts of the torque wrench

Fig. 1 Fig Torque wrench is composed as follows:

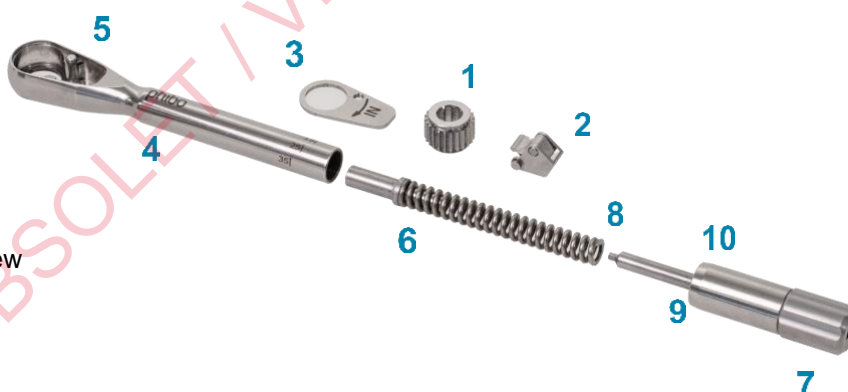- 1- Ratchet wheel
- 2- Ratchet
- 3- Cover
- 4- Ratchet head/handle
- 5- Screw
- 6- Spring/Stop
- 7- Torque adjustment screw
- 8- Hexagon shank bit
- 9- Bloster (PPS)
- 10- Brake (PTFE)

## Cleaning the torque wrench

When used in situations that do lead to operative residues (blood, secretions, tissue remnants), the torque wrench must be disassembled completely (fig 2 to 4) and placed in a suitable bath of disinfection in accordance with the recommendations of the manufacturer.

This operation facilitates cleaning because dry residues cause corrosion.

After cleaning, thoroughly rinse the parts with water and use a nylon brush to rub internal and external surfaces of the various parts of the torque wrench. During the cleaning process, avoid all contact between each part of the torque wrench.

## Disassembling the torque wrench

*Fig.2 Slide spring out of the handle*

### Fig. 2

Unscrew completely the torque adjustment screw (7) and extract the spring/stop (6). If necessary, use the hexagon shank bit (8) to extract the spring. Be careful not to damage the hexagon shank bit. Do not separate the stop from the spring (6).

*Fig. 3 Unscrew the screw (5) with the hexagon shank bit*

### Fig. 3

With the hexagon shank bit (8) of the torque adjustment screw (7), unscrew the screw (5), while maintaining a light pressure, to remove the cover (3).

*Fig. 4 Take out the ratchet and the ratchet wheel from the ratchet head*

### Fig. 4

The ratchet the torque head is composed of two removable elements: the ratchet (2) and the ratchet wheel (1). The screw (5) is fixed inside ratchet head.

## Assembling the torque wrench

*Fig. 5 Inset the ratchet wheel and the ratchet into the ratchet head*

### Fig. 5

To assemble the torque wrench, insert the next two elements in the following sequence: first the ratchet wheel (1) and then the ratchet (2).

*Fig. 6 Lubricating point*

### Fig. 6

Moderately lubricate the functional contact area between the teeth of the ratchet wheel (1) and the pivot point of the ratchet (2) as shown in fig 6.

Remove any trace of the lubricant from the outer surface of the torque wrench because excessive lubricant leaves traces on the instrument's surface during sterilisation.

Use only the instrument lubricant delivered with the torque wrench.

### Fig. 7

Once the parts (1) and (2) are in place, put the cover (3) on the ratchet head (4) and adjust. Turn the torque wrench while holding the cover. Fix the screw (5) tightly with hexagon shank bit (7) of the torque adjustment screw.

*Fig. 8 Insert spring/stop inside the handle and screw the torque adjustment screw*

**Fig. 8**

The assembly of the torque wrench is complete as soon as the spring/stop is in place and the torque adjustment screwed back in the handle.

Check the function by activating the ratchet wheel. If there is a regular ratchet noise, the instrument operates correctly.

Before use always check the assembly and the correct function of the torque wrench.

*Fig. 9. Use the adjustment key (provided) to obtain the higher torque values more easily.*

**Sterilisation**

The instrument must undergo a sterilisation with steam at 134°C/273°F during 18 minutes.

Apply the regulations in force in the country where the torque wrench is used. Before sterilisation, the torque wrench must be completely assembled (fig.5 to 8). Sterilise the key according to cycles of sterilisation recommended equipped with a vacuum pump (type B) to decrease the risk of formation of air pockets. This recommendation is particularly important for hollow instruments and guarantees a perfect drying.

We advise against the use of a hot air steriliser because it can lead to ageing of the spring and subsequently bring about a change of the torque value.

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